

Instructions for Use:

Read instructions prior to use.

1. Description and Intended Use

Description: The EcoPad Small IV is a biodegradable dermal pad with perforations for easy use.

Specifications: 0.5" x 8.0" x 11.0" biodegradable polyurethane foam pad with perforations. Perforations allow pad to divide into 4 rectangular 5.5" x 4.0" pads. Each smaller pad has a removable oval (described as "Oval").

2. Instructions for Use

Preparation: Remove the pad from the package prior to use. Keep pad in a clean and free of debris and other contaminants.

General Use Instructions: To provide adequate patient padding and protection this device can be used as a full-size pad or separated into smaller pads along any perforated line (see illustrations).

For use in padding peripheral IV's, monitoring cables, and pressure points.





The EcoPad Small IV is intended for single patient use only.

3. Storage and Handling

Storage Conditions: For best shelf life, store in original packaging at room temperature. Keep in humidity-controlled environment, <60% humidity.

Handling Instructions: After removal from packaging, use precautions to keep pad as clean as possible. Follow Instruction for Use (Section 2) for best handling practices.

4. Disposal: Disposal of this device should be in accordance with hospital guidelines.

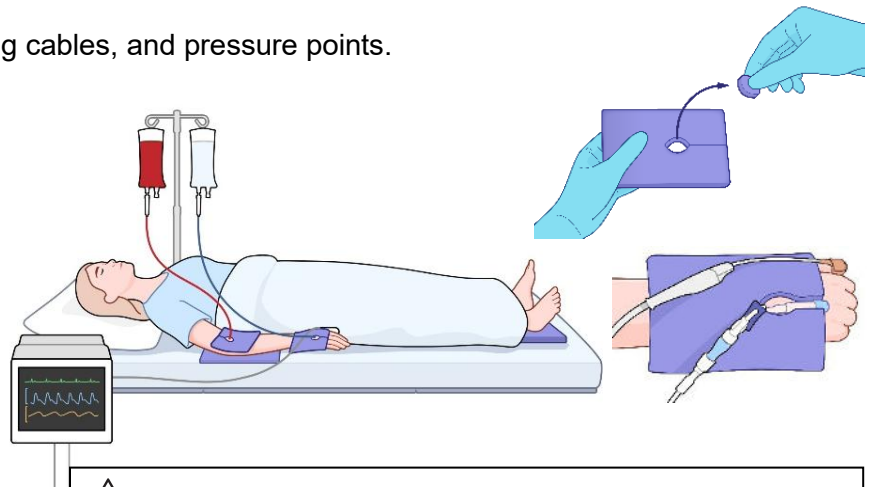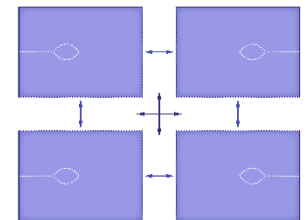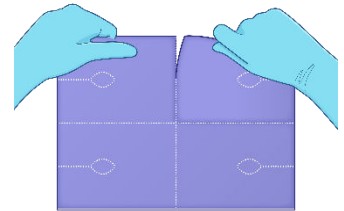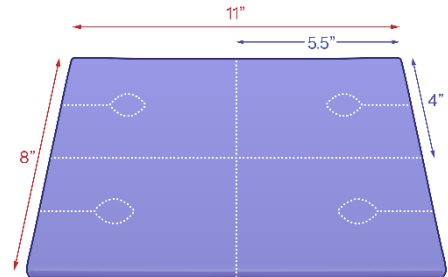

⚠ Caution:

- Always follow your facility's safe patient handling policies and procedures.
- This device has removable oval pieces that could become a hazard. Avoid use next to open wounds where unremoved ovals could dislodge into an open wound or mouth.

⚠ Warning:

- This device is not packaged as sterile and is NOT to be used as a wound dressing in post-surgical or treatment procedures.
- This device should NOT be used or placed in direct contact with open wounds.